


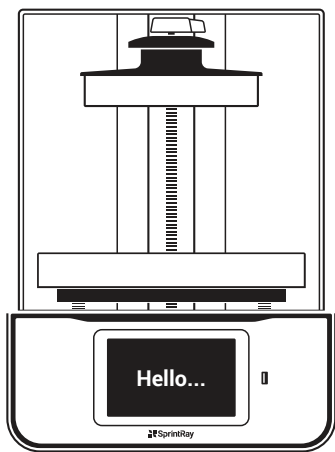

SprintRay

Pro S

Desktop 3D Printer Designed for Digital Dentistry
Quick Start Guide

Congratulations!

In this brief guide, we're going to show you how to set up and make your first model on the new SprintRay® Pro S Desktop 3D Printer.

Please follow the instructions carefully and visit our website for tips & tricks.

We're so excited to see what you make!

Training and Support

Contact our team of support technicians with questions.




Visit support.sprinray.com
Call (0) 800-70708000
Email support.eu@sprinray.com

1 In the Box


The print platform and resin tank are pre-installed and have a protective film that must be removed prior to first use.



The print platform and resin tank will be a different size depending on the model of your printer.

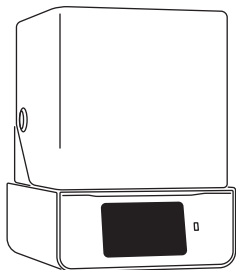


Pro S 3D Printer
Pro 55S / Pro 95S

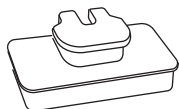

Print Platform
Pre-installed




Resin Tank and Cover
Pre-installed

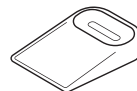


Resin Wiper




Power Cable

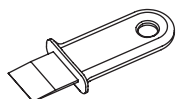

Print Removal Tool

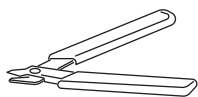

Support Snipper

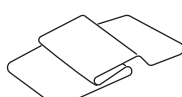

Cleaning Cloth




Onboarding Card


2 Where to Set Up

Situate your Pro 3D printer in a well-ventilated area on a flat and level surface.


Keep it away from extreme temperatures, windows, bright UV light sources, and direct sunlight.

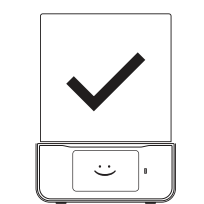

Level Surface

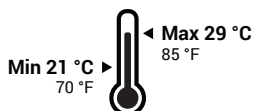

**Recommended
Operating Temperature**

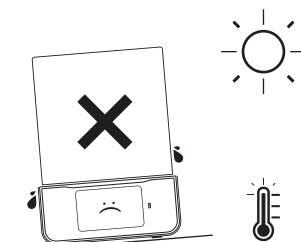


Uneven Surface / Warm Environment

3 Plug in Pro S

Use the provided power cable to plug your printer into a surge protector or uninterrupted power supply.

When not in use, the printer automatically goes into lower power mode, and does not need to be turned off.

Use the power switch if you need to perform a hard reset or disconnect your Pro S from its power source.

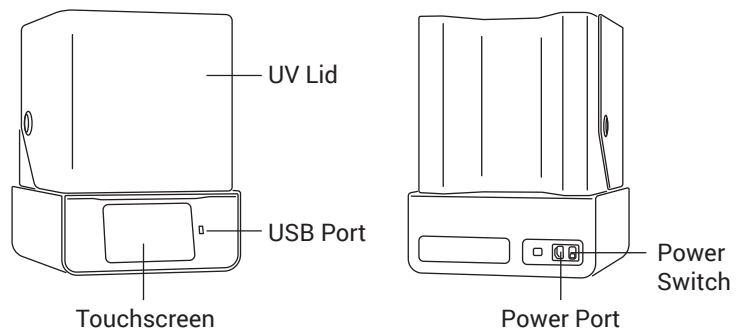

4 Set up Pro S

After powering-up your Pro S, you will be presented with onboarding instructions on the screen which will help you connect to the internet.



Make Printer Discoverable

Ensure that your printer and computer are both connected to the same wireless network.




5 Prepare Resin Tank

(Pre-installed)



Lock: Place the tank in its cradle, then push away from you. You will hear a 'click' when the tank is properly locked.

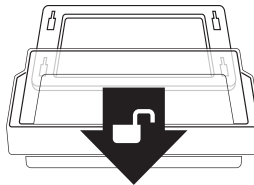

Unlock: Gently pull toward you until you hear a 'click' to unlock. Lift up to remove the tank from its cradle.



Smart Life Tracking

Your SprintRay Pro S is equipped with advanced electronics that allow the printer to track the lifetime and health of each tank. Follow onscreen instructions to properly set up new resin tanks to ensure longevity and proper tracking.

Resin Tanks Are Consumable

Resin tanks are consumable products that have an expected lifetime. Follow the onscreen instructions to understand the remaining life of your resin tank.



Backup Resin Tank: Keep a backup resin tank on hand to avoid downtime due to tank life.

Resin Tank Fill Lines

When starting a print, the desktop software or touchscreen of your Pro S 3D printer will display the resin level necessary for the planned job.

Fill the tank to the suggested level, ensuring even distribution of the resin across the tank. Pay attention to the Min and Max indications in the tank to ensure that the correct amount of resin is present. Before printing, double check resin levels against the value needed which is displayed on the screen.

The Min/Max indicators on the Pro 55S and Pro 95S resin tank are in different locations and indicate different amounts.


Resin Tank Best Practices

For best results, use a dedicated resin tank for each different material you use. Keep a backup tank on hand in case of an emergency to avoid ordering downtime.



WARNING!

Over pouring resin will cause spillage that is not covered by the limited warranty offered by SprintRay.

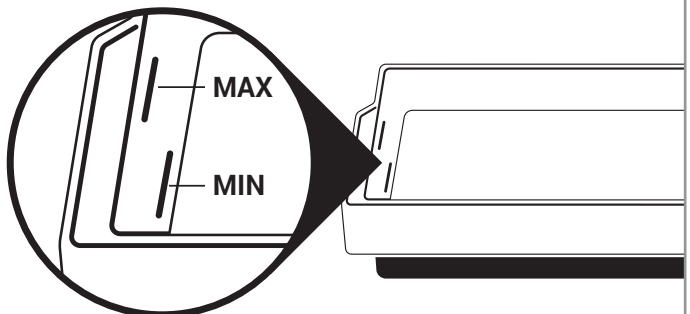


Pro 95S Min 200mL
Max 500mL

Pro 55S Min 150mL
Max 400mL



Stir Resin Before Print

If you keep resin in the tank between jobs, use the resin wiper to mix the tank before every print.

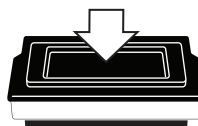

Cover Resin Tank After Print

If you're done printing for the day, secure the tank cover to help extend the life of resin in tank.

6

Prepare Print Platform


(Pre-installed)

Platform Heating

SprintRay Pro S is equipped with a heated build platform to improve printing reliability. The platform may be warm to the touch when this feature is enabled.




Built-in Heater

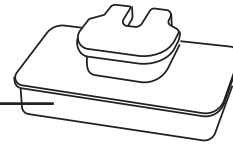


Locking/Unlocking the Print Platform


The SprintRay Pro S print platform uses magnetic locking. When the platform is seated on the Z-axis arm, activate the magnetic lock by turning the red knob clockwise. Turn the knob counter-clockwise to unlock the platform before removal.

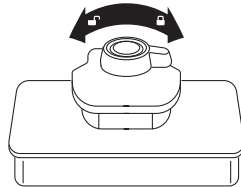

Turn to lock/unlock.

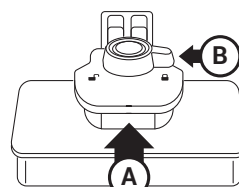

Insert and lock.

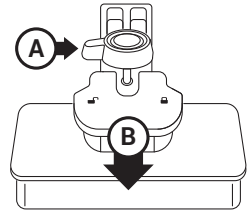

Unlock and pull to remove.

7

Get Software

Download RayWare Software from sprintray.com/software



Compatible with
Mac and Windows

8

3D Printing

Preparing the Pro S 3D Printer

Install resin tank and print platform. It is extremely important to make sure both the print platform and resin tank are fully secured and locked before you start a print.



Wear Gloves

We strongly recommend wearing gloves while handling printer and resin.




Inspect and Stir Gently

Inspect resin tank and mix resin with resin wiper before every print.



Gently Shake Resin Bottle to Mix


Before you pour resin into your Pro's tank, it is important to shake the bottle until it is thoroughly mixed.

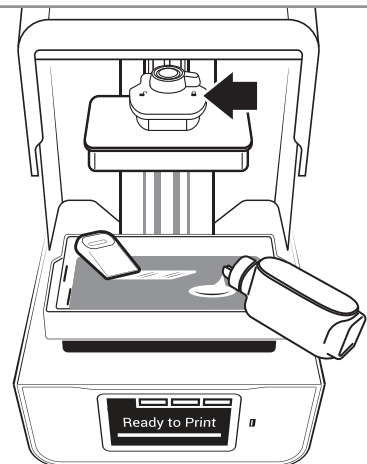

Prepare a Print Job in RayWare

After launching RayWare, select your printing material and the desired layer thickness. Import your STL model, orient the model, and add supports if necessary.

The model is now ready for printing. Click on the green printer icon, select your 3D printer from the list of available printer(s) and click Print.

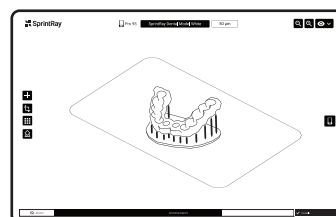

- 1 Select printer/resin type
- 2 Import Model(s)
- 3 Orient Model(s)
- 4 Add supports (if needed)
- 5 Send to printer

The software will process the job and send it to your printer.



Make Printer Discoverable

Ensure that your printer and computer are both connected to the same wireless network.



Firewall/Antivirus May Block Connection

Keep in mind that firewall or antivirus software may prevent your computer from finding the printer. Consult SprintRay customer support if you need help with connection issues.

Printing


The print platform should now slowly descend into the resin tank. Once this has happened, the print process has begun.

Depending on the type of resin you are printing with, the printer may have to go through a warming resin process. Warming resin may take up to 15 minutes.

Your printer runs independently once it begins a print job. Any changes to your network or computer will not affect print progress as long as the printer maintains power.



Keep the UV lid closed during the entire print cycle.


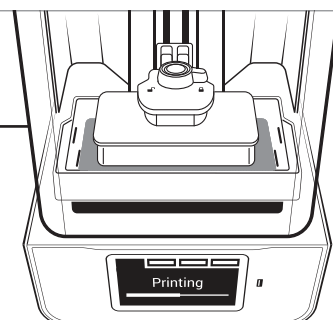


9 Finishing a Print Job

Removing Prints from Print Platform

When the print job is complete and the build arm has fully raised, remove the platform and place it on a flat surface.

Using the provided print removal tool, detach the part from print platform. Using the support snipper, remove the supports from your part.

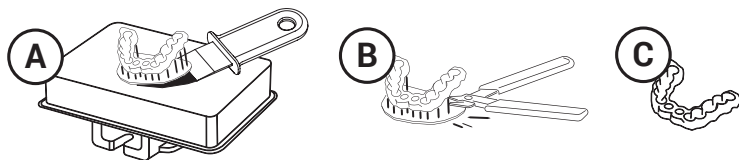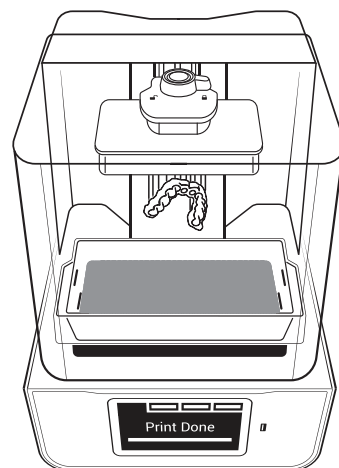


Wear Gloves

We strongly recommend wearing gloves while handling printer and resin.



Minimize Clean Up

To minimize cleanup, always remove the platform before removing your resin tank.

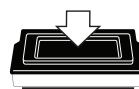


Cover Resin Tank After Print

If you're done printing for the day, secure the tank cover to help extend the life of resin in tank.

Washing Models with SprintRay Pro Wash/Dry™

Use IPA 99% to wash the 3D printed models prior to post curing. To ensure complete cleanliness, the washing process can be repeated. We recommend using SprintRay Pro Wash/Dry to perform a fully-automated, two-stage wash and dry.


Start washing models by simply placing the print platform in the Pro Wash/Dry to wash models and the platform simultaneously.

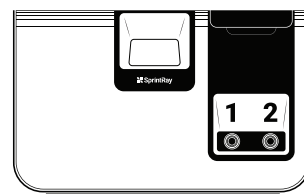

Post Curing Models with SprintRay ProCure 2™

Depending on the material specifications and regional regulatory requirements, you may need to post-cure the printed part after wash. You can use presets on SprintRay ProCure 2 to complete the post-curing in a few minutes.


SprintRay Pro Wash/Dry and ProCure 2 are sold separately.

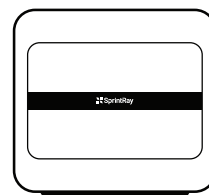

10 Maintenance

Clean 3D Printer Optical Glass Regularly

If resin cures onto the optical window, spray the glass with IPA and wipe with a soft cloth. Repeat if needed. Spray with glass cleaner and use a soft cloth to clean away fingerprints or other smudges.

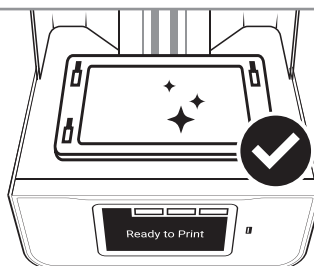


Cleaning the Printer Chassis

Clean the printer chassis with a soft, damp cloth. If resin is spilled on the printer, clean up immediately and wipe clean with a paper towel and IPA.

Clean Underside Tank Glass Regularly

Avoid touching the glass on the underside of the resin tank. To clean, spray with IPA to remove resin and then use glass cleaner to finish.

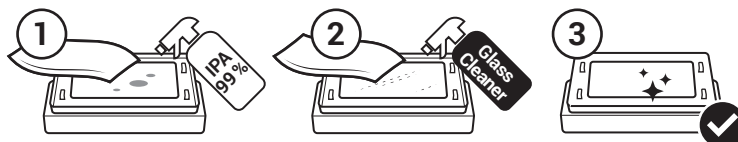

Backup Resin Tank

Resin tanks are consumable products that have an expected lifetime. Keep a backup resin tank on hand to avoid downtime due to tank life.

Software Updates

SprintRay will periodically publishes software updates for the Pro S 3D printer. Ensure your printer is connected to internet to receive software updates.

Resin Care

Cured resin can be discarded as regular trash. Never dispose of liquid resin in your sink or other plumbing.

Reorder Consumables

To reorder consumables, including materials and resin tanks, contact your equipment dealer representative.