




Multi-P® - Multi Purpose

THE MULTI – P® is a pre-fixed appliance used for patients between the ages of 9 to 13 years old. It offers eruption guides for the incisors, cuspids, and bicuspids. Options for a “low volume” or “high volume” are available. The “low volume” appliance is indicated for mesofacial or brachyfacial skeletal patterns, while the “high volume” is indicated for dolichofacial skeletal patterns.

Material: Silicone

Competition:
Ortho-Tain Nite-Guide®

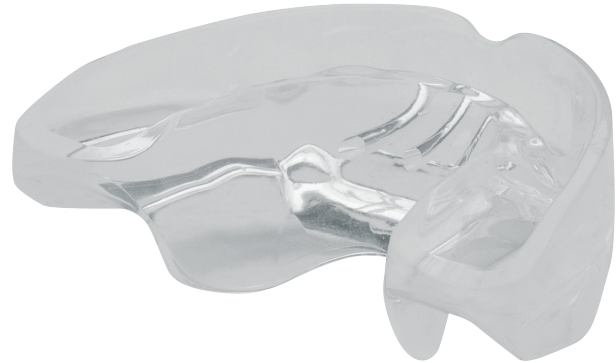

- Children ages 9-13 years old
- Eruption guides for incisors, cuspids, and bicuspids
- Available in two sizes: "low volume" and "high volume"

Type	Age	Sizes	Holes	Lip-Bumper Effect
Multi-P®	9-13	multiple	yes	no

Type	Guidance
Multi-P®	Incisors, Canines and Bicuspids

Multi-P® Sizer	Part Number	Package Contains
	J00980	1 each

Multi-P®: Multi - Positioner Appliance					
Low Model		High Model			
Size Number	Part Number	Size Number	Part Number	Maxillary Measurement (mm)	Mandibular Measurement (mm)
10	J00810			24.8 - 25.9	18.8 - 19.5
15	J00815			26 - 27.1	19.6 - 20.3
20	J00820	20	J00920	27.2 - 28.3	20.4 - 21.1
25	J00825	25	J00925	28.4 - 29.5	21.2 - 21.9
30	J00830	30	J00930	29.6 - 30.7	22 - 22.7
35	J00835	35	J00935	30.8 - 31.9	22.8 - 23.5
40	J00840	40	J00940	32 - 33.1	23.6 - 24.3
45	J00845	45	J00945	33.2 - 34.3	24.4 - 25.1
50	J00850	50	J00950	34.4 - 35.5	25.2 - 25.9
55	J00855	55	J00955	35.6 - 36.7	26 - 26.7
60	J00860	60	J00960	36.8 - 37.9	26.8 - 27.5
65	J00865	65	J00965	38 - 39.1	27.6 - 28.3
70	J00870	70	J00970	39.2 - 40.3	28.4 - 29.1

Multi-Family™ Frequently Asked Questions

1. How do you clean it?

-Daily:

- o Before use rinse under running water
- o After use, wash the appliance with a toothbrush under running water (Toothpaste can be used occasionally if necessary. As toothpaste is mildly abrasive, excessive use of toothpaste may scratch the surface.)

-When needed:

- o Disinfect the appliance in boiling water
- o Wash the box in a dishwasher
- o Sterilization in autoclave at 273 F

2. How long does the patient have to wear it?

- As for fixed appliances, it is impossible to give an exact time for how long treatment will take. It is a very complex question because many factors come into play:

- o Type/severity of malocclusion
- o Skeletal growth pattern of patient
- o Patient specific individual characteristics of dental development
- o How early/late is treatment initiated
- o Clinical preferences of the clinician
- o Patient compliance

