



The Multi-TB® - Multi Trainer for Braces


Multi-TB® - Multi Trainer for Braces is suitable for children or adults 13 years and older. Because it has no eruption guides it can be used with conventional fixed appliances. The Multi-TB® facilitates mandibular growth and helps unlock the TMJ.

Material: Silicone

Competition

Ortho-Tain Nite-Guide®

- Children ages 9-13 years old
- Eruption guides for incisors, cuspids, and bicuspids
- Available in two sizes: "low volume" and "high volume"

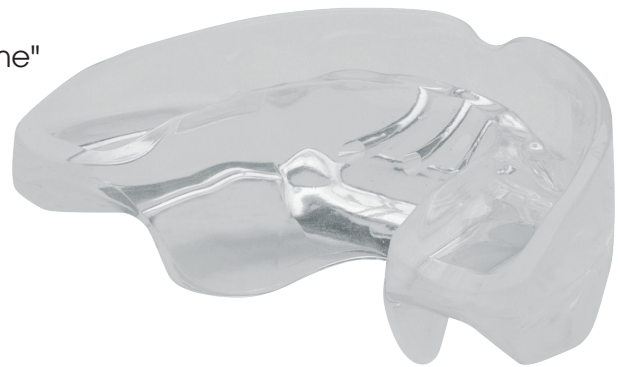

Type	Age	Sizes	Holes	Lip-Bumper Effect
Multi-TB®	all	1	no	yes

Type	Guidance
Multi-TB®	No guidance

	Part Number	Package Contains
Multi-TB®: Multi - Trainer for Brackets Appliance	J00990	1 each
	J00991	5 each

Multi-Family™ Frequently Asked Questions

1. How do you clean it?

-Daily:

- o Before use rinse under running water
- o After use, wash the appliance with a toothbrush under running water (Toothpaste can be used occasionally if necessary. As toothpaste is mildly abrasive, excessive use of toothpaste may scratch the surface.)

-When needed:

- o Disinfect the appliance in boiling water
- o Wash the box in a dishwasher
- o Sterilization in autoclave at 273 F

2. How long does the patient have to wear it?

- As for fixed appliances, it is impossible to give an exact time for how long treatment will take. It is a very complex question because many factors come into play:

- o Type/severity of malocclusion
- o Skeletal growth pattern of patient
- o Patient specific individual characteristics of dental development
- o How early/late is treatment initiated
- o Clinical preferences of the clinician
- o Patient compliance

